GLOSS-DERMA Pro

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Automatic application of serum with adjustable liquid output

GLOSS-DERMA uses essence liquid to penetrate the skin automatically when working.

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GLOSS-DERMA is an innovation technology in the field of aesthetic, it's also a new generation of MTS brand founded by Ekai in 2014. GLOSS-DERMA takes not only the advantages from dermaplaner and dermapen, more importantly, which works with its patent cartridge to apply the serum or essence automatically.

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Adjustable liquid output
In order to adapt to essence with different consistency, there are three levels of customized liquid output, and the liquid output can be controlled with the speed gear. The higher the speed gear, the more liquid output frequency, and vice versa.

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Speed 3
Fine lines around the eyes
スピード3: 目元の小じわケア

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Due to the effect of force, the higher the speed gear, the more the frequency of liquid discharge and vice versa.

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NO.1 with low battery warning helps users to know the battery remaining power intuitively, Better control the working hours.

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Page 6

Three kind of needle types solve different skin problems
* Needles are disposable, safe and sanitary

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異なる肌の悩みに対応
Automatic application of serum
When the cartridge acts on the skin up and down, the microneedle pierces the epidermis and opens thousands of fine channels. At this time, the essence liquid will automatically penetrate from the applicator and reach the deep skin evenly, with the absorption rate up to 90% higher than that of manual application. With the function of microneedle, the skin tissue repair can be quickly started, stimulate cell regeneration, restore the vitality and elasticity of the skin, and double the healing effect.

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