# GLOSS-DERMA Pro

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**GLOSS-DERMA Pro**

**Automatic application of serum with adjustable liquid output**

GLOSS-DERMA uses essence liquid to penetrate the skin automatically when working.

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GLOSS-DERMA is an innovation technology in the field of aesthetic, it's also a new generation of MTS brand founded by Ekai in 2002. GLOSS-DERMA takes not only the advantages from dermroller and dermapen, more importantly, which works with its potent cartridge to apply the serum or essence automatically.

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**Adjustable liquid output**  
In order to adapt to essence with different consistency, there are three levels of customized liquid output, and the liquid output can be controlled with the speed gear. The higher the speed gear, the more liquid output frequency, and vice versa.

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Gloves are required when taking out the cap to adjust the liquid output

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3ml capacity  
Designed for a complete facial care  
容量:3ml

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Speed 3  
Fine lines around the eyes  
スピード3: 目元の小じわケア

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Due to the effect of force, the higher the speed gear, the more the frequency of liquid discharge and vice versa.

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**NO.1 with low battery warning helps users to know the battery remaining power intuitively, Better control the working hours.**

Intuitive low battery warning  
低バッテリー警告

The number of speed flashes slowly when the battery capacity has 10% left

The pen will turn off automatically after the number of speed flashes quickly for 1 minutes.

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The pen shows "L" when restarting after the low battery automatic shutdown.

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Page 6

**Three kind of needle types solve different skin problems**  
\* Needles are disposable, safe and sanitary

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**Nano-HS**  
For fine lines around eyes, sensitive muscles, red blood

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**Automatic application of serum**

When the cartridge acts on the skin up and down, the microneedle pierces the epidermis and opens thousands of fine channels. At this time, the essence liquid will automatically penetrate from the applicator and reach the deep skin evenly, with the absorption rate up to 90% higher than that of manual application. With the function of microneedle, the skin tissue repair can be quickly started, stimulate cell regeneration, restore the vitality and elasticity of the skin, and double the healing effect.

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Purchase Date / 購入日 \_\_\_\_\_ / \_\_\_\_ / \_\_\_\_

Warranty Period / 保証期間 From purchase date: \_ 1 \_ Year(s)

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